

101.865-12 – including *Taq* polymerase

101.865-12u – without *Taq* polymerase

Lot No.: 5E7

Olerup SSP® HLA-C*15 Add-on

Product number: 101.865-12 – including *Taq* polymerase
101.865-12u – without *Taq* polymerase
Lot number: 5E7
Expiry date: 2019-05-01
Number of tests: 12
Number of wells per test: 3+1

CHANGES COMPARED TO THE PREVIOUS HLA-C*15 ADD-ON LOT (91X):

Well	5'-primer	3'-primer	rationale
4	-	-	Updated Negative Control.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

The HLA-C*15:02, 15:05, 15:29 and 15:87 alleles give different patterns in the HLA-C*15 Add-on kit¹; www.ebi.ac.uk/imgt/hla, 2016-July-14, release 3.25.0.

The HLA-C*15 Add-on kit cannot distinguish the silent mutations in the C*15:02:01:01-15:02:26 or the C*15:05:01-15:05:12 alleles.

¹Alleles that have been deleted from or renamed in the official WHO HLA Nomenclature up to and including the last IMGT/HLA database release can be retrieved from web page <http://hla.alleles.org/alleles/deleted.html>.

RESOLUTION IN HOMO- AND HETEROZYGOTES:

Good.

INFLUENCE ON THE INTERPRETATION OF HLA-C*15 ADD-ON SUBTYPINGS BY NON-HLA-C*15 ALLELES:

None.