

Olerup SSP[®] HLA-C*17

Product number: 101.628-06 – including *Taq* polymerase
101.628-06u – without *Taq* polymerase
Lot number: 64K
Expiry date: 2013-April-01
Number of tests: 6
Number of wells per test: 8

CHANGES COMPARED TO THE PREVIOUS HLA-C*17 LOT (29G):

The primers of the wells detailed below have been exchanged, added or modified compared to the previous lot.

Well	5'-primer	3'-primer	rationale
1	-	Added	Primer added for the C*17:01:04 allele.
7	New	New	New primer pair for the C*17:06 allele.
8	New	New	New primer pair for the C*17:07 allele.

THE NUMBER OF WELLS has been increased from 6 to 8.

ALLELE COVERAGE:

C*17:01 to C*17:07, i.e. all the currently recognized HLA-C*17 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2010-July-16, release 3.1.0.

RESOLUTION IN HLA-C*17 HOMO- AND HETEROZYGOTES:

Excellent.

INFLUENCE ON THE INTERPRETATION OF HLA-C*17 SUBTYPINGS BY NON-HLA-C*17 ALLELES:

Non of importance.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.