

Olerup SSP[®] HLA-Cw*01

Product number: 101.621-12 – including *Taq* polymerase
101.621-12u – without *Taq* polymerase
Lot number: 68F
Expiry date: 2011-March-01
Number of tests: 12
Number of wells per test: 16

CHANGES COMPARED TO THE PREVIOUS HLA-Cw*01 LOT (44E):

Well	5'-primer	3'-primer	rationale
8	-	Added	New primer for the Cw*0122 allele.
10	-	Added	New primer for the Cw*0122 allele.
13	Removed	Removed	Primer pair removed for improved resolution.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

Cw*0102 to Cw*0122 i.e. all the currently recognized HLA-Cw*01 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2009-January-16, release 2.24.0.

RESOLUTION IN HLA-Cw*01 HOMO- AND HETEROZYGOTES:

The Cw*0102,0102 genotype give rise to a unique amplification pattern.

INFLUENCE ON THE INTERPRETATION OF HLA-Cw*01 SUBTYPINGS BY NON-HLA-Cw*01 ALLELES:

None.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.