

-TAC

HLA-Cw*08
101.623-12 – including *Taq* polymerase
101.623-12u – without *Taq* polymerase

Product Insert

Page 1 of 1

Lot No.: **23E**

Lot-specific information

Olerup SSP[®] HLA-Cw*08

Product number: 101.623-12 – with *Taq* polymerase
101.623-12u – without *Taq* polymerase

Lot number: 23E

Expiry date: 2010-April-01

Number of tests: 12

Number of wells per test: 16

CHANGES COMPARED TO THE PREVIOUS HLA-Cw*08 LOT (X25):

Well	5'-primer	3'-primer	rationale
5	New	New	Primer pair added for the Cw*0815 allele.
15	Modified	-	Changed 5'-primer to decrease the tendency to form primer dimer artefacts.
16	Modified	-	Modified primer added for increased yield of specific PCR product.

THE NUMBER OF WELLS is unchanged.

ALLELE COVERAGE:

Cw*0801 to Cw*0816, i.e. all the currently recognized HLA-Cw*08 alleles, give rise to unique amplification patterns; www.ebi.ac.uk/imgt/hla, 2008-April-08, release 2.21.0.

RESOLUTION IN HLA-Cw*08 HOMO- AND HETEROZYGOTES:

The Cw*0801,0801 and the Cw*0801,0816 genotypes give rise to identical amplification pattern.

INFLUENCE ON THE INTERPRETATION OF HLA-Cw*08 SUBTYPINGS BY NON-HLA-Cw*08 ALLELES:

None frequently occurring.

MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.