

## Olerup SSP<sup>®</sup> HLA-Cw\*17

**Product number:** 101.628-06 – including *Taq* polymerase  
101.628-06u – without *Taq* polymerase  
**Lot number:** 29G  
**Expiry date:** 2011-September-01  
**Number of tests:** 6  
**Number of wells per test:** 6

### CHANGES COMPARED TO THE PREVIOUS HLA-Cw\*17 LOT (69E):

The primers of the wells detailed below have been exchanged, added or modified compared to the previous lot.

Well	5'-primer	3'-primer	rationale
6	New	New	New primer pair for the Cw*1705 allele.

**THE NUMBER OF WELLS** has been increased from 5 to 6.

### ALLELE COVERAGE:

Cw\*1701 to Cw\*1705, i.e. all the currently recognized HLA-Cw\*17 alleles, give rise to unique amplification patterns; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2009-July-17, release 2.26.0.

### RESOLUTION IN HLA-Cw\*17 HOMO- AND HETEROZYGOTES:

Excellent.

### INFLUENCE ON THE INTERPRETATION OF HLA-Cw\*17 SUBTYPINGS BY NON-HLA-Cw\*17 ALLELES:

No influence by non-HLA-Cw\*17 alleles..

### MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:

No comments received.