

## **Olerup SSP<sup>®</sup> HLA-Cw\*06**

**Product number:** 101.614-12 – including *Taq* polymerase  
101.614-12u – without *Taq* polymerase  
**Lot number:** 01F  
**Expiry date:** 2010-July-01  
**Number of tests:** 12  
**Number of wells per test:** 16

### **CHANGES COMPARED TO THE PREVIOUS HLA-Cw\*06 LOT (X85):**

<b>Well</b>	<b>5'-primer</b>	<b>3'-primer</b>	<b>rationale</b>
9	Added	Added	Primer pair added for the Cw*0617 allele.

**THE NUMBER OF WELLS** is unchanged.

### **ALLELE COVERAGE:**

Cw\*0601 to Cw\*0617, i.e. all the currently recognized HLA-Cw\*06 alleles, give rise to unique amplification patterns; [www.ebi.ac.uk/imgt/hla](http://www.ebi.ac.uk/imgt/hla), 2008-April-08, release 2.22.0.

### **RESOLUTION IN HLA-Cw\*06 HOMO- AND HETEROZYGOTES:**

The Cw\*0602,0602 genotype gives rise to a unique amplification pattern.

### **INFLUENCE ON THE INTERPRETATION OF HLA-Cw\*06 SUBTYPINGS BY NON-HLA-Cw\*06 ALLELES:**

None frequently occurring.

### **MODIFICATIONS MADE DUE TO COMMENTS FROM CUSTOMERS:**

No comments received.